

# Find-Me Tunstall Watch

Introducing the next generation  
**Find-Me Tunstall Carers Watch**  
 with GPS tracking and fall  
 detection.



The Find-Me Tunstall Carers Watch reduces the risk to an individual by alerting trained response operators of their location when in need of assistance.

A cellular personal emergency response system, panic alert, GPS tracker and fall detector all in one, the watch offers an unobtrusive duress alert device for those who may feel vulnerable in the community.

Monitored by Tunstall's 24-hour response centre, the Find-Me Tunstall Carers Watch can be tailored to suit the individual and easily adapted as their needs change.

## How does it work?

Located on the side of the watch, the alert button is programmed to contact Tunstall's 24-hour monitoring centre.

When the alert button is pressed, it connects to the monitoring centre, putting the wearer in contact with a trained operator who will assess the situation and contact the necessary responder to assist, such as a carer, family member or emergency services.

Using GPS location details and real-time tracking, the care consultant is able to accurately locate the wearer and direct their responders to the precise location.

With open voice communication, the wearer can also speak directly with the operator.

Each watch is supplied with a pre-configured SIM card and will operate where there is mobile network coverage.

## Who is it for?

The Find-Me Tunstall Carers Watch is designed to assist:

- Older people
- People living with dementia
- Those who want extra reassurance while at home, work and out in the community
- People living with a disability
- Those who need to send a silent duress alert
- People living with chronic disease
- People who require immediate emergency assistance in a duress situation
- People who work alone, travel regularly, or work in potentially hazardous environments



# Find-Me Tunstall Carers Watch

Go anywhere, any time with the Find-Me Tunstall Carers Watch.



## Features and benefits

- Provides an easy to use duress button
- Can be used outside of the home, where there is mobile network coverage
- Programmable safe zone, to raise an alert when the wearer leaves a specified area
- Destination alert, to advise when the wearer reaches a programmed destination
- Open voice communication, allowing the wearer to speak directly with a response operator
- GPS tracking to accurately locate the wearer
- Map location and street address provided on the activation of an alert
- NEW** water resistant enabling it to be worn in the rain and shower
- NEW** automatic alert generated on detection of falls and watch removal
- NEW** auto-presence alerting to ensure the watch is active and in-range
- NEW** magnetic battery charging technology

## Standards

<b>RCM</b>	Compliant
	AS/CA S042.1-2015, AS/CA S042.4-2015
	AS/NZS 60950-1: 2011
	EN 55022:2010
	EN624979: 2010, EN62209-2: 2010 (body), EN503860: 2001/A1: 2012
<b>CE:</b>	Compliant
<b>Warranty:</b>	12 months

## Technical details

<b>Weight:</b>	<100 gms (including battery)
<b>Dimensions:</b>	60 x 46 x 15 mm (without strap)
<b>Battery type:</b>	Li-ion rechargeable
<b>Battery usage:</b>	15 hours (typical use)
<b>Battery life:</b>	2 years +
<b>Device class:</b>	Cellular personal emergency response system
<b>Device communication:</b>	Voice / data / SMS
<b>Frequency:</b>	850 / 2100
<b>Compatibility:</b>	3G / NextG
<b>Water resistance:</b>	IPX4 splash resistant

Our policy of continual development means that product specifications and appearance may change without notice. Tunstall does not accept responsibility for any errors and omissions contained within this document.

© 2016 Tunstall Australasia

**Tunstall Australasia**  
1800 603 377  
www.tunstallhealthcare.com.au

**Tunstall New Zealand**  
0800 488 678  
www.tunstall.co.nz

